

# Tubifast® 2-Way Stretch®


A high quality  
tubular bandage for  
patients' complete  
freedom of movement

Contours to any shaped  
limb or torso thanks to  
2-Way Stretch® Technology

Washable and re-usable

Self-retaining tubular construction:  
no pins or tapes are needed

Easy to use: can be cut to  
required length

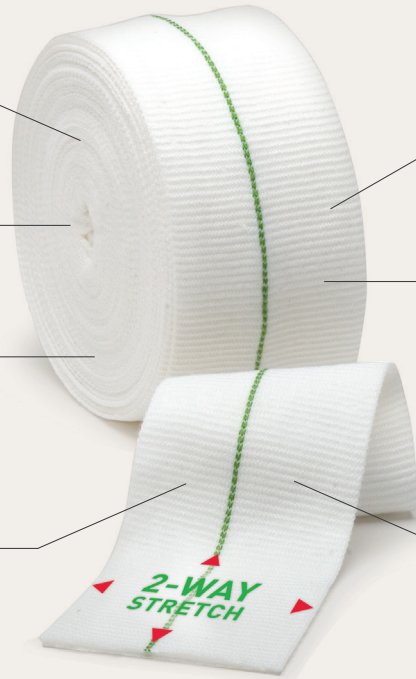

Latex free


Light elasticity in both vertical  
and horizontal directions  
allowing patients of complete  
freedom of movement<sup>1</sup>

Available in 5 colour-coded  
widths to fit any part of the body


Tubifast® 2-Way Stretch® tubular bandages are designed for dressing retention patch wrapping (wet or dry) and as an undercast stockinette. Tubifast 2-Way Stretch can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa<sup>2</sup>.

- Allows complete freedom of movement
- Self-retaining with no pins or tapes necessary
- Reusable after washing


# Instructions for use Tubifast® 2-Way Stretch®

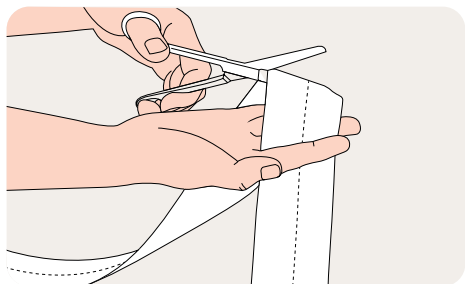

1. Choose the correct bandage width, using the quick reference colour coding, and cut to length.

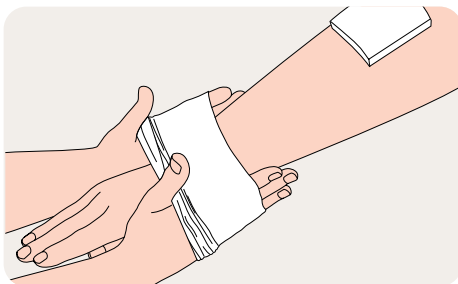

2. Stretch Tubifast® 2-Way Stretch® over the affected area.

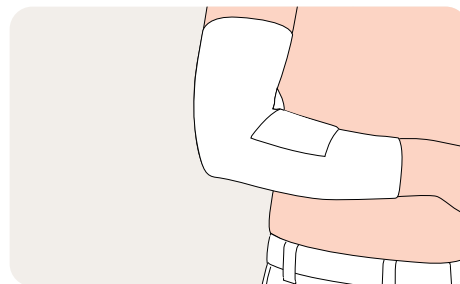

3. Position Tubifast 2-Way Stretch to cover the dressing. Illustrated, step-by-step instructions are available for more advanced applications.

## How Tubifast® 2-Way Stretch® works

Tubifast 2-Way Stretch is a self-retaining tubular construction that allows air to circulate over the skin. Tubifast 2-Way Stretch provides a light elasticity in both vertical and horizontal directions allowing patients of complete freedom of movement.

## Benefits of Tubifast 2-Way Stretch

- Contours to any shaped limb or torso thanks to 2-Way Stretch® technology
- Allows complete freedom of movement
- Easy to use: can be cut to required length
- Self-retaining with no pins or tapes necessary
- Latex free, non-sensitising and non-irritating
- Reusable after washing
- Available in 5 colour-coded widths to fit any part of the body

## Areas of use

Tubifast 2-Way Stretch tubular bandages are designed for dressing retention without the need for pins or tapes; it is particularly suitable for holding dressings in place on difficult areas such as stumps of amputee patients.

The product is also designed for patch wrapping (wet or dry) and as an undercast stockinette. Tubifast 2-Way Stretch can also be used for skin covering for any part of the body and in the management of Epidermolysis Bullosa<sup>2</sup>.

## Washing instructions



## Fibre content

Viscose, Elastane and Polyamide.



## Tubifast 2-Way Stretch ordering information

Product	Width x Length	Limb	Art no.	PIP Code	NHS Code
<b>Red line</b> Small limbs	3.5cm x 1m	9-18cm	12480	005-0088	-
	3.5cm x 10m	9-18cm	2434	026-8698	EGP115
<b>Green line</b> Small and medium limbs	5cm x 1m	14-24cm	2481	005-0039	-
	5cm x 3m	14-24cm	2485	255-1885	-
	5cm x 5m	14-24cm	3555	255-5803	-
	5cm x 10m	14-24cm	2436	026-8680	EGP117
<b>Blue line</b> Large limbs	7.5cm x 1m	24-40cm	2482	004-9924	-
	7.5cm x 3m	24-40cm	2486	255-1877	-
	7.5cm x 5m	24-40cm	3556	255-5811	-
	7.5cm x 10m	24-40cm	2438	026-6924	EGP119
<b>Yellow line</b> Extra-large limbs, heads, children's trunks	10.75cm x 1m	35-64cm	2483	023-6539	-
	10.75cm x 3m	35-64cm	2487	255-1869	-
	10.75cm x 5m	35-64cm	3557	255-5829	-
	10.75cm x 10m	35-64cm	2440	000-6221	EGP121
<b>Purple line</b> Adult trunks	20cm x 1m	64-130cm	2479	324-1452	-
	20cm x 5m	64-130cm	3558	324-1486	-
	20cm x 10m	64-130cm	2444	-	EGP107

Tubifast 2-Way Stretch is packaged in single packs.

**References:** 1. Eytier C, Gazeau E, Beneteau G, Verfaillie G. Convenience and tolerance of the combination of a soft silicone foam dressing and a 2-way stretch tubular bandage in the management of local wounds. *Journal des plaies et cicatrisations* 2013;18(88):38-44.  
2. Mölnlycke Health Care. Data on file.

## Find out more at molnlycke.com

Mölnlycke Health Care AB, P.O. Box 6, Entreprenörstråket 21, SE-431 21 Mölndal Sweden. Phone + 46 31 722 30 00.  
The Mölnlycke, Tubifast and 2-Way Stretch trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2025 Mölnlycke Health Care AB. All rights reserved. HQIM006537

