


Prolapse ring Incontinence

PERK00 - PERK07

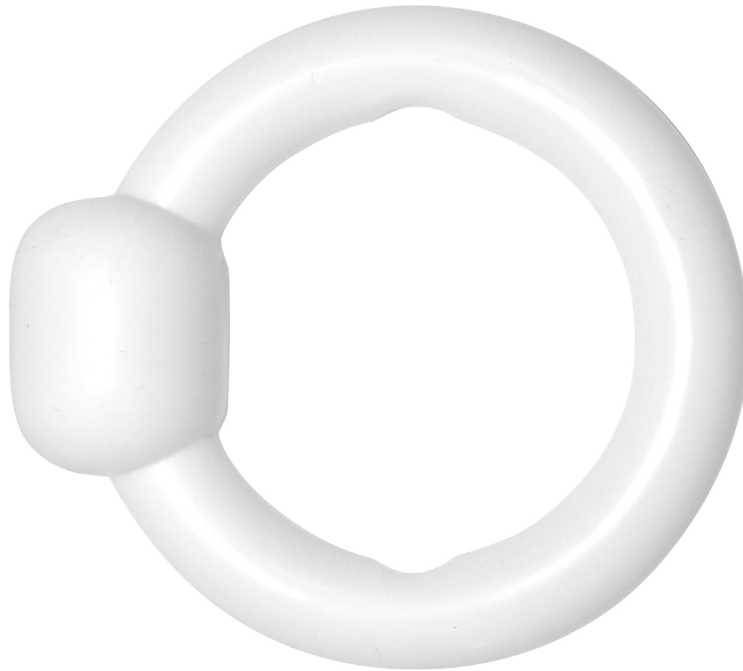

Prolapse ring Inkontinence

The flexible silicone Prolapse ring is commonly used for a mild first or second degree uterine prolapse or procidentia.

- The Prolapse ring are made from 100% medical grade silicone.
- The Prolapse rings are designed for single patient use.
- Fitting usually requires a trial of various sizes to determine the proper Prolapse ring size.
- A vaginal lubricant may be required for the patient's comfort before insertion of the Prolapse ring.
- The physician may suggest K-Y jelly®, or equivalent.


The physician should determine if the patient requires oestrogen therapy before prescribing a pessary. Within 24 to 48 hours, ensure the patient is not allergic to the pessary. Examine the vagina and ask the patient if there has been any discomfort, irritation, pressure, sensitivity, or unusual vaginal discharge. Also determine if there has been any improvement in her personal symptoms.

Guangzhou Fame Medical Co., Ltd. is a company specializing in the R&D, production, and sales of medical silicone products. Founded in 2007, Fame Medical has been dedicated to the R&D and production of medical silicone products for over ten years, possessing a profound understanding of silicone products and their technical processes. Fame Medical can provide comprehensive production process training and product processing services, including product development, mold design, and medical silicone accessories.

Material Silicone



| | |
|--------|---|
| PERK00 | Prolapse ring Incontinence 43 mm (9616-0) |
| PERK01 | Prolapse ring Incontinence 49 mm (9616-1) |
| PERK02 | Prolapse ring Incontinence 56 mm (9616-2) |
| PERK03 | Prolapse ring Incontinence 62 mm (9616-3) |
| PERK04 | Prolapse ring Incontinence 68 mm (9616-4) |
| PERK05 | Prolapse ring Incontinence 74 mm (9616-5) |
| PERK06 | Prolapse ring Incontinence 81 mm (9616-6) |
| PERK07 | Prolapse ring Incontinence 87 mm (9616-7) |

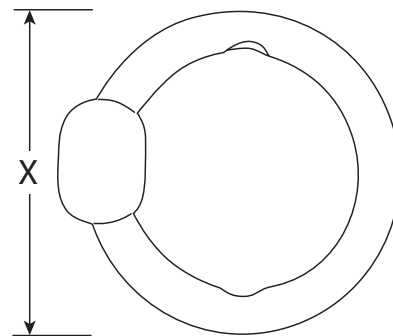

| ARTICLE NO. | DEPT. PACKAGING |
|---------------|-----------------|
| PERK00-PERK07 | 1 pcs |