



PRODUCT TECHNICAL FORM

Device name : Grasping Forceps

Trade name: /

Description: Grasping Forceps is classified as Class I with Sterilization according to Rule 5 with sterilization in Annex VIII of REGULATION (EU) 2017/745.

References: See Annex 1 Product List (MTN-0025241)

Intended Use: Grasping Forceps are used with an endoscope to take out foreign bodies from the digestive tract and the respiratory tract.

| | |
|------------------------------------|--|
| Intended patient population | The product is intended for adult and adolescent populations. |
| Security of use | <p>The proposed device Grasping Forceps is a sterile, single-use endoscope device, which is intended to take out foreign bodies from the digestive tract and the respiratory tract. It is intended for use by physicians or medical personnel who have received appropriate training, in accordance with the applicable laws.</p> <p>The Grasping Forceps is composed of Jaw Assembly, Spring Sheath and Handle Assembly. The slider moved back and forth on the mandrel which let the jaws open and close, by opening and closing the jaws to achieve the purpose of grasping the foreign bodies.</p> |
| Recommendations | After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy. |



PRODUCT TECHNICAL FORM

Device Description:

PHOTOS, DRAWINGS

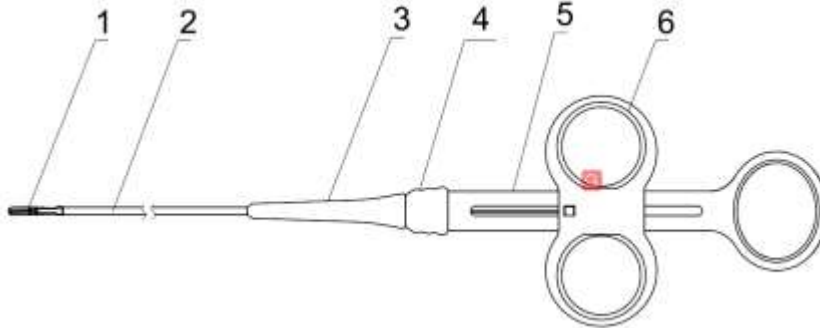


Figure 1 – Assembly Drawing

| No | Part Name | No | Part Name |
|----|------------------------|----|--------------|
| 1 | Jaws Assembly | 4 | Position Cap |
| 2 | Spring Sheath | 5 | Mandrel |
| 3 | Spring Protecting Tube | 6 | Slider |

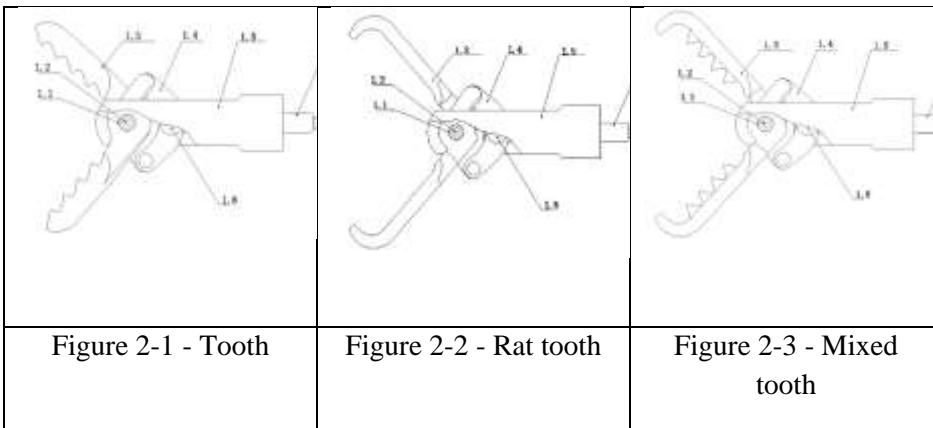


Figure 2 Jaws Assembly Drawing

| No | Part Name | No | Part Name |
|-----|----------------|-----|-------------|
| 1.1 | Shim | 1.5 | Housing |
| 1.2 | Rivet I | 1.6 | Tension Rod |
| 1.3 | Jaw | 1.7 | Drive Wire |
| 1.4 | Connecting Rod | 1.8 | Rivet IV |

PHOTOS, DRAWINGS
Product Illustration

Device Illustration – Proposed devices in Pouch



Coated



Uncoated

Device Illustration – Proposed devices without package



Coated



Uncoated

Device Illustration –open



Mixed tooth forceps



Rat tooth forceps



Tooth forceps



PRODUCT TECHNICAL FORM

Invasive device / non invasive device: Invasive device

Long term / short term: Transient

Components: materials of components

1. Product

| No. | Component | Material | Biodegradable | Human or Animal Origin | Contact Human |
|-----|------------------------|-------------|---|---|---|
| 1 | Jaws Assembly | | | | |
| 1.1 | Shim | SUS304 | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.2 | Rivet I | SUS303F | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.3 | Jaw | 17-4PH | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.4 | Connecting Rod | SUS304 | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.5 | Housing | SUS303F | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.6 | Tension Rod | SUS304 | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.7 | Drive Wire | SUS304 | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 1.8 | Rivet IV | SUS303F | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 2 | Spring Sheath | SUS304+HDPE | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No |
| 3 | Spring Protecting Tube | TPE | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No |
| 4 | Position Cap | ABS757 | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No |
| 5 | Mandrel | ABS757+TPE | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No |
| 6 | Slider | ABS757+TPE | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No | <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No |

2. Packaging

- A. The pouch: ARJO, PE/PET
- B. BOX: Paper Single wall corrugated fiberboard
- C. Carton: Paper Double wall corrugated fiberboard
- D. Overshipper: Paper Double wall corrugated fiberboard

| Presence of: | YES | NO | | YES | NO |
|--|-----|-------------------------------------|-------------------|-----|-------------------------------------|
| Active substances | | <input checked="" type="checkbox"/> | Phtalate and DEHP | | <input checked="" type="checkbox"/> |
| PVC: | | <input checked="" type="checkbox"/> | Bisphenol A | | <input checked="" type="checkbox"/> |
| Product of animal or biological origin | | <input checked="" type="checkbox"/> | Latex | | <input checked="" type="checkbox"/> |



PRODUCT TECHNICAL FORM

Manufacturer

| | |
|---------------------------------------|--|
| Manufacturer name | Micro-Tech (Nanjing) Co., Ltd. |
| Manufacturing location(s) | No.10 Gaoke Third Road, Nanjing National Hi- Tech, Industrial Development Zone Nanjing 210032, Jiangsu Province PEOPLE'S REPUBLIC OF CHINA |
| Manufacturer Quality Assurance System | EN ISO 13485: 2016 |

Sterilisation

| | |
|---------------------------|--|
| Sterilisation method | EO sterile |
| Sterilisation location(s) | No. 199 Medicine Valley Avenue, Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, PRC. |
| Sterilisation control(s) | Annual Sterilization Validation, Routine Sterilization Parameters Control and BI Testing |
| Shelf Life | 5 years |

| | |
|--------------------|---|
| Storage conditions | The product should be stored in a cool, dry, clean, well-ventilated, non-corrosive gas environment. The product is valid for 5 years. Prior to use, check the expiry date of sterilization on the label of the package. |
|--------------------|---|

| | |
|---|--|
| CE Mark Number | CE 0123 |
| Notified body | TÜV SÜD Product Service GmbH |
| European representative (full name, address and unique identification) | Shanghai International Holding Corp. GmbH(Europe) Eiffestrasse 80, 20537 Hamburg Germany |
| Class of the device (I,IIa,IIb,III) | I with sterilization |
| EMDN Code | G03080102 |
| GMDN Code | 33199 |



PRODUCT TECHNICAL FORM

Labelling:

| Pouch label | Case Box label | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
|--|---|---|--|-----------------------|--------------------|------------|-----------------|------------|--------------------|-----------------|--------------------|-----------------|--------------------|-----------------|--------------------|-----------------|---|---|--|--|---|---|--|-----------------------|--------------------|------------|-----------------|------------|---|---|--|--------------------|-----------------|------------|--------|--|--------|
| <p style="text-align: center;">PHOTO</p> <p style="text-align: center;">Grasping Forceps</p> <table style="width:100%; font-size: small;"> <tr> <td>Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане</td> <td>Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão</td> <td>Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid</td> <td>Hvatalica Griptáng</td> </tr> </table> <p style="text-align: right; font-size: x-large;">1 PC</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;"> <p>6.3 mm</p> </div> <div style="text-align: center;"> <p>2.3 mm</p> <p>Diameter</p> </div> <div style="text-align: center;"> <p>2.8 mm</p> <p>Working channel</p> </div> <div style="text-align: center;"> <p>230 cm</p> <p>Working length</p> </div> </div> <table border="1" style="width:100%; font-size: x-small; margin-top: 10px;"> <tr> <td>REF NGF01-11023230</td> <td>2025-01-20</td> </tr> <tr> <td>LOT ME111111111</td> <td>2030-01-19</td> </tr> </table> <div style="font-size: x-small; margin-top: 10px;"> <p>eifu.micro-tech.com.cn CAUTION_DESTROY IT AFTER USING</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>UDI (01)06902284544543(17)300119(10)ME111111111</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>Grasping Forceps</p> <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <td>REF NGF01-11023230</td> <td>LOT ME111111111</td> </tr> </table> <p>(01)06902284544543(17)300119(10)ME111111111</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>Grasping Forceps</p> <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <td>REF NGF01-11023230</td> <td>LOT ME111111111</td> </tr> </table> <p>(01)06902284544543(17)300119(10)ME111111111</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>Grasping Forceps</p> <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <td>REF NGF01-11023230</td> <td>LOT ME111111111</td> </tr> </table> <p>(01)06902284544543(17)300119(10)ME111111111</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>Grasping Forceps</p> <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <td>REF NGF01-11023230</td> <td>LOT ME111111111</td> </tr> </table> <p>(01)06902284544543(17)300119(10)ME111111111</p> </div> <p style="text-align: center; font-weight: bold;">EUNGF01-11023230 P-100008506 0 01/25</p> <div style="font-size: x-small; margin-top: 10px;"> <p>Micro-Tech (Nanjing) Co., Ltd. No. 10 Gaoke Third Road, Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, PRC Tel: +86-25-58609879, 58646393 Fax: +86-25-58744269 E-mail: info@micro-tech.com.cn Http://www.micro-tech-medical.com</p> <table style="width:100%; font-size: x-x-small;"> <tr> <td>Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany</td> <td>MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland</td> <td>UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom</td> </tr> </table> </div> | Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане | Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão | Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid | Hvatalica Griptáng | REF NGF01-11023230 | 2025-01-20 | LOT ME111111111 | 2030-01-19 | REF NGF01-11023230 | LOT ME111111111 | REF NGF01-11023230 | LOT ME111111111 | REF NGF01-11023230 | LOT ME111111111 | REF NGF01-11023230 | LOT ME111111111 | Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany | MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland | UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom | <p style="text-align: center;">PHOTO</p> <p style="text-align: center;">Grasping Forceps</p> <table style="width:100%; font-size: small;"> <tr> <td>Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане</td> <td>Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão</td> <td>Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid</td> <td>Hvatalica Griptáng</td> </tr> </table> <p style="text-align: right; font-size: x-large;">10 PCS</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;"> <p>6.3 mm</p> </div> <div style="text-align: center;"> <p>2.3 mm</p> <p>Diameter</p> </div> <div style="text-align: center;"> <p>2.8 mm</p> <p>Working channel</p> </div> <div style="text-align: center;"> <p>230 cm</p> <p>Working length</p> </div> </div> <table border="1" style="width:100%; font-size: x-small; margin-top: 10px;"> <tr> <td>REF NGF01-11023230</td> <td>2025-01-21</td> </tr> <tr> <td>LOT ME111111111</td> <td>2030-01-20</td> </tr> </table> <div style="font-size: x-small; margin-top: 10px;"> <p>eifu.micro-tech.com.cn CAUTION_DESTROY IT AFTER USING</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>UDI (01)16902284544540(17)300120(10)ME111111111</p> </div> <div style="font-size: x-small; margin-top: 10px;"> <p>Micro-Tech (Nanjing) Co., Ltd. No. 10 Gaoke Third Road, Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, PRC Tel: +86-25-58609879, 58646393 Fax: +86-25-58744269 E-mail: info@micro-tech.com.cn Http://www.micro-tech-medical.com</p> <table style="width:100%; font-size: x-x-small;"> <tr> <td>Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany</td> <td>MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland</td> <td>UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom</td> </tr> </table> </div> <p style="text-align: center; font-weight: bold;">EUNGF01-11023230 P-100008507 0 01/25</p> <div style="font-size: x-small; margin-top: 10px;"> <p>Grasping Forceps</p> <table style="width:100%; font-size: x-small;"> <tr> <td>REF NGF01-11023230</td> <td>LOT ME111111111</td> </tr> <tr> <td>2030-01-19</td> <td>2.3 mm</td> </tr> <tr> <td></td> <td>230 cm</td> </tr> </table> <p>Diameter Working length</p> </div> | Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане | Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão | Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid | Hvatalica Griptáng | REF NGF01-11023230 | 2025-01-21 | LOT ME111111111 | 2030-01-20 | Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany | MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland | UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom | REF NGF01-11023230 | LOT ME111111111 | 2030-01-19 | 2.3 mm | | 230 cm |
| Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане | Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão | Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid | Hvatalica Griptáng | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | 2025-01-20 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| LOT ME111111111 | 2030-01-19 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | LOT ME111111111 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | LOT ME111111111 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | LOT ME111111111 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | LOT ME111111111 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany | MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland | UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Forceps de prindere Prijemalka s kavčjem Kleszcze chwytające Pince de prehension Griebiančios žnyplės Satveršanas knaibles Щипчики за захващане | Úchopové klešče Úchopové klešče Pinzas de agarre Λαβίσα σύλληψης Kavrama Forsepəsi Pinzette da presa Pinças de Preensão | Grijptang Grijptang Faszszangen Fogócsipesz Gripende tang Tarttumispíhdít Haaramistangid | Hvatalica Griptáng | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | 2025-01-21 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| LOT ME111111111 | 2030-01-20 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Shanghai International Holding Corp. GmbH (Europe) Elfenstrasse 80, 20537 Hamburg, Germany | MedEnvoy Switzerland Golfhardstrasse 28, 6302 Zug, Switzerland | UK Responsible Person Lotus Global Co., Ltd. 23 Main Street, Reading, RG2 6AG, England, United Kingdom | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| REF NGF01-11023230 | LOT ME111111111 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| 2030-01-19 | 2.3 mm | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | 230 cm | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |



PRODUCT TECHNICAL FORM

| Carton label | Overshipper label |
|---|---|
| <p>PHOTO</p> <p>Grasping Forceps</p> <p>Forceps de prindere Prijemalka s kavljem Kleszcze chwytające Pince de préhension Griebiančios žnyplės Satveršanas knaibles Щипкиге за захвщане</p> <p>Úchopové kliešte Úchopové kliešte Pinzas de agarre Λαβίδα σύλληψης Kavrama Forsepsi Pinzette da presa Pinças de Preensão</p> <p>Grijptang Gripetang Fasszangen Fogócsipesz Gribende tang Tartumispidit Haaramistangid</p> <p>Hvatalica Griptång</p> <p>100 PCS</p> <p>6.3 mm</p> <p>2.3 mm</p> <p>Diameter</p> <p>2.8 mm</p> <p>Working channel</p> <p>230 cm</p> <p>Working length</p> | <p>PHOTO</p> <p>Grasping Forceps</p> <p>Forceps de prindere Prijemalka s kavljem Kleszcze chwytające Pince de préhension Griebiančios žnyplės Satveršanas knaibles Щипкиге за захвщане</p> <p>Úchopové kliešte Úchopové kliešte Pinzas de agarre Λαβίδα σύλληψης Kavrama Forsepsi Pinzette da presa Pinças de Preensão</p> <p>Grijptang Gripetang Fasszangen Fogócsipesz Gribende tang Tartumispidit Haaramistangid</p> <p>Hvatalica Griptång</p> <p>200 PCS</p> <p>6.3 mm</p> <p>2.3 mm</p> <p>Diameter</p> <p>2.8 mm</p> <p>Working channel</p> <p>230 cm</p> <p>Working length</p> |
| <p>REF NGF01-11023230</p> <p>2025-01-21</p> | <p>REF NGF01-11023230</p> <p>2025-01-21</p> |
| <p>LOT ME111111111</p> <p>2030-01-20</p> | <p>LOT ME111111111</p> <p>2030-01-20</p> |
| <p>UDI (01)26902284544547(17)300120(10)ME111111111</p> <p>STERILEEO MD CE 0123</p> <p>eifu.micro-tech.com.cn CAUTION, DESTROY IT AFTER USING</p> <p>Micro-Tech (Nanjing) Co., Ltd. No. 10 Gaoke Third Road, Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, PRC Tel: +86-25-58609879, 58646393 Fax: +86-25-58744269 E-mail: info@micro-tech.com.cn Http://www.micro-tech-medical.com</p> <p>EC REP Shanghai International Holding Corp. GmbH (Europe) Elifstrasse 80, 20537 Hamburg, Germany</p> <p>CH REP MedEnvoy Switzerland Gotthardstrasse 28, 6302 Zug, Switzerland</p> <p>UK Responsible Person Lotus Global Co., Ltd. 23 Maine Street, Reading, RG2 6AG, England, United Kingdom</p> <p>EUNGF01-11023230 P-100008507 0 01/25</p> | <p>UDI (01)36902284544544(17)300120(10)ME111111111</p> <p>STERILEEO MD CE 0123</p> <p>eifu.micro-tech.com.cn CAUTION, DESTROY IT AFTER USING</p> <p>Micro-Tech (Nanjing) Co., Ltd. No. 10 Gaoke Third Road, Nanjing National Hi-Tech Industrial Development Zone, Nanjing 210032, Jiangsu Province, PRC Tel: +86-25-58609879, 58646393 Fax: +86-25-58744269 E-mail: info@micro-tech.com.cn Http://www.micro-tech-medical.com</p> <p>EC REP Shanghai International Holding Corp. GmbH (Europe) Elifstrasse 80, 20537 Hamburg, Germany</p> <p>CH REP MedEnvoy Switzerland Gotthardstrasse 28, 6302 Zug, Switzerland</p> <p>UK Responsible Person Lotus Global Co., Ltd. 23 Maine Street, Reading, RG2 6AG, England, United Kingdom</p> <p>EUNGF01-11023230 P-100008507 0 01/25</p> |
| <p>Barcode type</p> | <p>UDI (01)36902284544544(17)300120(10)ME111111111</p> |



PRODUCT TECHNICAL FORM

Packaging:
Type:

| | |
|-------------------|-----------------|
| Sterile packaging | Box of 10 units |
| <p>PHOTO</p> | <p>PHOTO</p> |

| | |
|--------------------------|--------------------------|
| Casebox box of 100 units | Overshipper of 200 units |
|--------------------------|--------------------------|

| | |
|--------------|--------------|
| <p>PHOTO</p> | <p>PHOTO</p> |
|--------------|--------------|

| | |
|--|--|
| Simple or double | 1 piece simple |
| Number of units per Non-sterile packaging unit | 1pc/Pouch, 10 pcs/ case box, 100 pcs/ carton, 200 pcs/ Overshipper |
| Final packaging | Overshipper |
| Product conservation | Packed as the packaging. |



PRODUCT TECHNICAL FORM

| Revision | Date | Description |
|-----------------|-------------|--------------------|
| A/0 | 2025-03-26 | Initial |



Specification list of Grasping Forceps (MDR)

MTN-0025241

Applicable area: EU

受控

| 受控状态 Status | 2021-03-02 | 版本 Rev. | A/0 |
|----------------|---|--------------|------------|
| | 职能 Position | 签名 Signature | 日期 Date |
| 编写 Draft | 设计工程师 Design Engineer | 徐晓青 | 2021-03-01 |
| 审核 Review | 研发技术负责人 Person Responsible for R&D Technology | 倪平 (M) | 2021-03-01 |
| 批准 Approval | 部门经理 Manager of General Consumable Manufacturing Dept. | 李王蕊 | 2021-03-02 |





Specification list of Grasping Forceps (MDR)

MTN-0025241

| Revision | Date | Description |
|----------|------------|-------------|
| A/0 | 2021-03-01 | Initial |

文件
021-03
生



Specification list of Grasping Forceps (MDR)

MTN-0025241(A/O)

| No. | REF | Classification of Handle | Classification of jaws | Classification of spring tube | Diameter Of Jaws (unit: mm) | Effective length (unit: mm) |
|-----|----------------|--------------------------|------------------------|-------------------------------|-----------------------------|-----------------------------|
| 1 | NGF01-11018120 | New handle | Rat tooth forceps | uncoated | 1.8 | 1200 |
| 2 | NGF01-11018160 | New handle | Rat tooth forceps | uncoated | 1.8 | 1600 |
| 3 | NGF01-11018180 | New handle | Rat tooth forceps | uncoated | 1.8 | 1800 |
| 4 | NGF01-11018230 | New handle | Rat tooth forceps | uncoated | 1.8 | 2300 |
| 5 | NGF01-11023120 | New handle | Rat tooth forceps | uncoated | 2.3 | 1200 |
| 6 | NGF01-11023160 | New handle | Rat tooth forceps | uncoated | 2.3 | 1600 |
| 7 | NGF01-11023180 | New handle | Rat tooth forceps | uncoated | 2.3 | 1800 |
| 8 | NGF01-11023230 | New handle | Rat tooth forceps | uncoated | 2.3 | 2300 |
| 9 | NGF03-11018120 | New handle | Rat tooth forceps | PE coating | 1.8 | 1200 |
| 10 | NGF03-11018160 | New handle | Rat tooth forceps | PE coating | 1.8 | 1600 |
| 11 | NGF03-11018180 | New handle | Rat tooth forceps | PE coating | 1.8 | 1800 |
| 12 | NGF03-11018230 | New handle | Rat tooth forceps | PE coating | 1.8 | 2300 |
| 13 | NGF03-11023120 | New handle | Rat tooth forceps | PE coating | 2.3 | 1200 |
| 14 | NGF03-11023160 | New handle | Rat tooth forceps | PE coating | 2.3 | 1600 |
| 15 | NGF03-11023180 | New handle | Rat tooth forceps | PE coating | 2.3 | 1800 |
| 16 | NGF03-11023230 | New handle | Rat tooth forceps | PE coating | 2.3 | 2300 |
| 17 | NGF11-11023120 | New handle | Tooth forceps | uncoated | 2.3 | 1200 |
| 18 | NGF11-11023160 | New handle | Tooth forceps | uncoated | 2.3 | 1600 |
| 19 | NGF11-11023180 | New handle | Tooth forceps | uncoated | 2.3 | 1800 |
| 20 | NGF11-11023230 | New handle | Tooth forceps | uncoated | 2.3 | 2300 |
| 21 | NGF13-11023120 | New handle | Tooth forceps | PE coating | 2.3 | 1200 |
| 22 | NGF13-11023160 | New handle | Tooth forceps | PE coating | 2.3 | 1600 |
| 23 | NGF13-11023180 | New handle | Tooth forceps | PE coating | 2.3 | 1800 |
| 24 | NGF13-11023230 | New handle | Tooth forceps | PE coating | 2.3 | 2300 |
| 25 | NGF31-11018120 | New handle | Mixed tooth forceps | uncoated | 1.8 | 1200 |
| 26 | NGF31-11018160 | New handle | Mixed tooth forceps | uncoated | 1.8 | 1600 |
| 27 | NGF31-11018180 | New handle | Mixed tooth forceps | uncoated | 1.8 | 1800 |
| 28 | NGF31-11018230 | New handle | Mixed tooth forceps | uncoated | 1.8 | 2300 |
| 29 | NGF31-11023120 | New handle | Mixed tooth forceps | uncoated | 2.3 | 1200 |
| 30 | NGF31-11023160 | New handle | Mixed tooth forceps | uncoated | 2.3 | 1600 |
| 31 | NGF31-11023180 | New handle | Mixed tooth forceps | uncoated | 2.3 | 1800 |
| 32 | NGF31-11023230 | New handle | Mixed tooth forceps | uncoated | 2.3 | 2300 |
| 33 | NGF33-11018120 | New handle | Mixed tooth forceps | PE coating | 1.8 | 1200 |
| 34 | NGF33-11018160 | New handle | Mixed tooth forceps | PE coating | 1.8 | 1600 |
| 35 | NGF33-11018180 | New handle | Mixed tooth forceps | PE coating | 1.8 | 1800 |
| 36 | NGF33-11018230 | New handle | Mixed tooth forceps | PE coating | 1.8 | 2300 |
| 37 | NGF33-11023120 | New handle | Mixed tooth forceps | PE coating | 2.3 | 1200 |
| 38 | NGF33-11023160 | New handle | Mixed tooth forceps | PE coating | 2.3 | 1600 |
| 39 | NGF33-11023180 | New handle | Mixed tooth forceps | PE coating | 2.3 | 1800 |
| 40 | NGF33-11023230 | New handle | Mixed tooth forceps | PE coating | 2.3 | 2300 |

微