

# Protefix® Active Cleanser

[Rate product](#)



- ✓ It removes food remains, coffee, tea and nicotine thoroughly
- ✓ It has a deodorising effect.
- ✓ Self-active cleansing within 15 minutes.

32 tablets

Available online and in your local pharmacy.

♥ WISH LIST

---

**Protefix® Active Cleanser has an antibacterial effect, removes plaque and 99,99% of bacteria.**

With the help of active oxygen, Protefix® Active Cleanser Tablets develop an excellent cleaning power without attacking the denture material. They are self acting and clean the denture quickly and thoroughly.

Additionally, Protefix Active Cleanser Tablets have deodorising qualities and ensure cleanliness and freshness throughout the day.

## Application & Composition

### Composition:

Potassium Caroate, Sodium Bicarbonate, Sodium Carbonate, Citric Acid, Sorbitol, VP/VA Copolymer, Sodium Lauryl Sulfate, Sodium Lauryl Sulfoacetate, Aroma, CI 73015

## Application:

### Dosage and mode of application

Dissolve one Protefix® Active Cleanser tablet in a glass of lukewarm water (100-200 ml, about 35°C) and place the denture or brace immediately in the glass. The denture or brace should be completely covered by the cleaning solution. Clean and fresh in 3 minutes, disinfected in 10 minutes. Cleaning also possible overnight. Remove the denture and rinse well under running water before replacing it in the mouth.

## Please note the following important advice:

Apply only outside of the mouth. Avoid contact with eyes. The product should be stored out of the reach of young children. Should a tablet or cleaning solution swallowed by mistake, please immediately seek medical advice. Store in a cool and dry place. Do not store above 25°C.

## Reviews

There are no reviews yet.

[WRITE A CUSTOMER REVIEW](#)

[↓ PACKAGE INSERT - PDF](#)

[♥ WISH LIST](#)

EAN: 4009932512335