

Protefix Adhesive Cream Extra Strong

with threefold effect



- ✓ Extra strong with effect on wet denture
- ✓ Extra strong with long-lasting effect. Gives the user confidence and independence
- ✓ an effective peripheral seal helps to prevent food residues getting underneath the denture

47 g cream

Available online and in your local pharmacy.

 [WISH LIST](#)

Time-proven formulation: without zinc, without preservatives. Also suited for partial dentures.

Protefix® Adhesive Cream Extra-Strong is the ideal solution for a wide range of adhesion problems. It is recommended for both difficult adhesion conditions and increased saliva flow. Protefix® Adhesive Cream Extra-Strong with threefold effect is activated immediately and gives the user confidence and wearing comfort for the whole day.

Protefix® Adhesive Cream Extra-Strong **with threefold effect:**

- **Effective on wet denture:** can be directly applied onto the moist denture.
- **Extra-Strong:** the long-lasting effect gives the user confidence and independence whether eating, drinking or laughing.
- **Lasting:** an effective peripheral seal helps to prevent food residues getting underneath the denture; the cushion effect enhances the wearing comfort.

Application & Composition

Composition:


Mixture of sodium and calcium salts of the copolymer of methyl vinyl ether and maleic acid anhydride, carboxymethyl cellulose, paraffin, vaseline, silica, menthol, azorubin.

Application:

Adhesive Cream for application on dentures.

Dosage and mode of application

Daily dosage: up to 0.8 g Adhesive Cream. Clean the dentures of any residue and dab Protefix® Adhesive Cream Extra-Strong on the wet denture (in the upper or lower denture recess that goes next to the jaw). If necessary, you may also apply a line of Protefix® Adhesive Cream Extra-Strong to the back of the upper plate. Then fit the denture and hold it firmly in place for a few seconds. To be used only on healthy mucous membranes. Wait 5 minutes before eating. You should apply Protefix® Adhesive Cream Extra-Strong sparingly because any excess can impair the adhesion of your denture. The use of Adhesive Cream should only be temporary. Have the fit of your dentures checked at regular intervals.

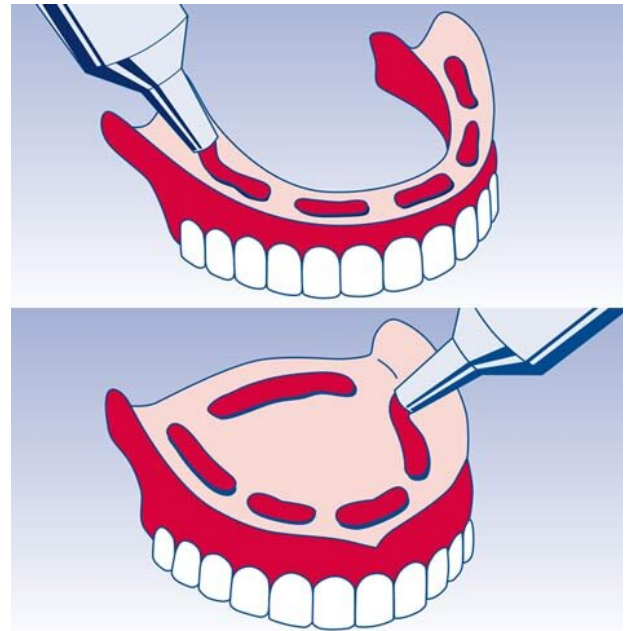

Contraindications

Hypersensitivity to one of the ingredients.

Side effects

In isolated cases, hypersensitivity reactions in the mucous membranes of the mouth or even systemically cannot be ruled out. If you suspect the presence of a hypersensitivity reaction, please contact your dentist.

Please note the following important advice:

- If you notice two phases in the product, shake and knead the tube before using.
- After applying the cream, seal the tube carefully, taking care to ensure that the opening remains dry and clean because any ingress of liquid may cause the adhesive cream to harden.
- Remove any residue of adhesive cream with a cotton-wool ball soaked in edible oil.
- Keep out of reach of children!
- Do not use after the expiry date.

Reviews