



Knowing now matters.™



Clearview[®] Mono

Two-step technology provides a quick, easy solution for diagnosing Infectious Mononucleosis.

The Clearview[®] Mono patented, innovative technology provides a highly sensitive and specific test for Infectious Mononucleosis (IM).

- Rapid Response: 4 easy steps, results in 5 minutes.
- Simplicity: CLIA waived*, easy-to-use test requires no training.
- Reliability: Built-in control ensures accuracy.
- Flexibility: Use with multiple sample types—whole blood, serum or plasma.

The Clearview[®] Mono provides a clear solution to the common challenge of diagnosing Infectious Mononucleosis (IM).

- Immediate therapy reduces time of discomfort for patient.
- Minimizes the risk of more serious illness.
- Reduces the spread of virus to others.



To learn more contact your local Account Executive | alere.com


Clearview® Mono Comparison Grid

Product Name	Clearview® Mono	Quidel QuickVue® + Infectious Mono	Genzyme OSOM® Mono
Format	20 Cassettes	20 Cassettes	25 Dipsticks
CLIA Status	Waived: Whole Blood Non-waived: Whole Blood, Serum or Plasma	Waived: Whole Blood Non-waived: Whole Blood, Serum or Plasma	Waived: Whole Blood Non-waived: Whole Blood, Serum or Plasma
Sample	Whole Blood (Fingerstick or Venipuncture), Serum or Plasma	Whole Blood (Fingerstick or Venipuncture), Serum or Plasma	Whole Blood (Fingerstick or Venipuncture), Serum or Plasma
Sensitivity	98%**	99.9%	100%
Specificity	>99%***	99.8%	90.3%
Procedural Steps	2	2	3
Time to Result	5 minutes	5 minutes	5 minutes
Shelf Life	18 months	18 months	12 months
Storage	Room Temperature	Room Temperature	Room Temperature


Procedure

1


2 drops Venipuncture whole blood + 1 drop Buffer

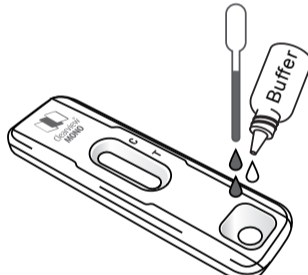

50 µL Fingerstick whole blood + 1 drop Buffer

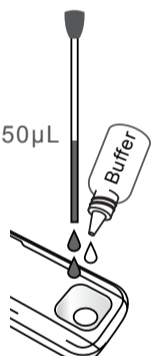


1 drop Serum or Plasma + 1 drop Buffer




OR OR

2


Wait 5 minutes, then Interpret Results




5
minutes

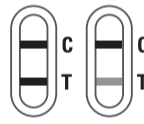

Positive



Negative

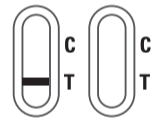

Invalid

*For whole blood specimens

**Positive agreement with comparative test

***Negative agreement with comparative test

© 2015 Alere. All rights reserved. The Alere Logo, Alere and Clearview are trademarks of the Alere group of companies. All other trademarks referenced are trademarks of their respective owners. The photos displayed are for illustrative purposes only. Any person depicted in such photographs is a model. 12000288E-03 2/15



Alere AB

tel: +46 8 544 812 10

www.alere.se

kundservice.se@alere.com

Order Information

Clearview® Mono

20 Test Kit

Order No. CV506815N

CLIA Waived: Whole Blood

Non-waived: Serum, Plasma