

VACUETTE® QUICKSHIELD Safety Tube Holders & Accessories:

Item. No.	Description	Packaging			
		Inner	Outer		
VACUETTE® QUICKSHIELD Safety Tube Holder					
450230	VACUETTE® QUICKSHIELD Safety Tube Holder	50 pcs.	1.000 pcs.		
VACUETTE® Multiple Use Drawing Needles (sterile)					
450071	VACUETTE® Multiple Use Drawing Needle, black	0.7 x 25mm	22G x 1"	100 pcs.	2.000 pcs.
450072	VACUETTE® Multiple Use Drawing Needle, green	0.8 x 25mm	21G x 1"	100 pcs.	2.000 pcs.
450073	VACUETTE® Multiple Use Drawing Needle, yellow	0.9 x 25mm	20G x 1"	100 pcs.	2.000 pcs.
450075	VACUETTE® Multiple Use Drawing Needle, black	0.7 x 38mm	22G x 1 1/2"	100 pcs.	2.000 pcs.
450076	VACUETTE® Multiple Use Drawing Needle, green	0.8 x 38mm	21G x 1 1/2"	100 pcs.	2.000 pcs.
450077	VACUETTE® Multiple Use Drawing Needle, yellow	0.9 x 38mm	20G x 1 1/2"	100 pcs.	2.000 pcs.
VACUETTE® VISIO PLUS Needles (sterile)					
450040	VACUETTE® VISIO PLUS Needle, green	0.8 x 38mm	21G x 1 1/2"	100 pcs.	2.000 pcs.
450041	VACUETTE® VISIO PLUS Needle, black	0.7 x 38mm	22G x 1 1/2"	100 pcs.	2.000 pcs.
450042	VACUETTE® VISIO PLUS Needle, green	0.8 x 25mm	21G x 1"	100 pcs.	2.000 pcs.
450043	VACUETTE® VISIO PLUS Needle, black	0.7 x 25mm	22G x 1"	100 pcs.	2.000 pcs.

These devices have no components made of natural dry rubber.

Your Power for Health


greiner bio-one



Your Power for Health


greiner bio-one



For further information, please visit our website www.gbo.com/preanalytics or contact us:

Austria (Main Office)

Greiner Bio-One GmbH
Phone +43 7583 6791-0
Fax +43 7583 6318
E-Mail office@at.gbo.com

Germany

Greiner Bio-One GmbH/Preanalytics
Tel +49 7022 948-0
Fax +49 7022 948-514
E-Mail office@de.gbo.com

Spain

VACUETTE Espana S.A.
Phone +34 91 652 77 07
Fax +34 91 652 33 35
E-Mail info@vacuette.es

United Kingdom

Greiner Bio-One Ltd.
Phone +44 1453 8252 55
Fax +44 1453 8262 66
E-Mail info@uk.gbo.com

Brazil

Greiner Bio-One Brasil
Phone +55 19 3468-9600
Fax +55 19 3468-9621
E-Mail office@br.gbo.com

Hungary

Greiner Bio-One Hungary Kft.
Phone +36 96 213 088
Fax +36 96 213 198
E-Mail office@hu.gbo.com

Switzerland

Greiner Bio-One VACUETTE
Schweiz GmbH
Phone +41 7 12 28 55 22
Fax +41 7 12 28 55 21
E-Mail office@ch.gbo.com

USA

Greiner Bio-One North America Inc.
Phone +1 704 261-7800
Fax +1 704 261-7899
E-Mail info@us.gbo.com

China

Greiner Bio-One Suns Co., Ltd.
Phone +86 10 83 55 19 91
Fax +86 10 63 56 69 00
E-Mail office@cn.gbo.com

India

Greiner Bio-One INDIA Pvt., Ltd.
Phone +91 120 456 8787
Fax +91 120 456 8788
E-Mail info@gboindia.com

Thailand

Greiner Bio-One Thailand Ltd
Phone +66 38 4656 33
Fax +66 38 4656 36
E-Mail office@th.gbo.com

France

Greiner Bio-One SAS
Phone +33 1 69 86 25 25
Fax +33 1 69 86 25 35
E-Mail office@fr.gbo.com

Netherlands

Greiner Bio-One B.V.
Phone +31 172 4209 00
Fax +31 172 4438 01
E-Mail info@nl.gbo.com



VACUETTE®
QUICKSHIELD
Safety Tube Holder

“Do the Twist“



980959 rev.00, 03.2014 e

VACUETTE® QUICKSHIELD Safety Tube Holder

Do the twist

More flexibility – as offered by the new **VACUETTE®** QUICKSHIELD Safety Tube Holder.

The safety shield of the newly designed holder can now be positioned in any direction. As per the slogan "Do the twist", right-handed and left-handed users can now rotate the safety shield to the position that suits them best.

The rotating safety shield ensures:

- clear visibility of the entire puncture site
- greater flexibility to adapt as effectively as possible to any situation

For daily blood collection procedures

The **VACUETTE®** QUICKSHIELD Safety Tube Holder from Greiner Bio-One is especially suitable for routine blood collection. There is no change to the usual collection technique, and the safety shield is activated either one-handed with the aid of a solid surface or with the thumb. This product can provide the user with the simplest handling and reliable infection protection.



The ergonomic design of the holder gives fingers a better grip for puncturing.

The safety shield is pre-assembled directly on the holder. The shield is to be manually activated upon removal of the needle from the patient's vein.



Without the need to change collection technique, the **VACUETTE®** QUICKSHIELD Safety Tube Holder helps prevent needlestick injuries.



VACUETTE® QUICKSHIELD can be used with either **VACUETTE®** Multiple Use Drawing Needles or **VACUETTE®** VISIO PLUS Needles. Both cannulas have a special needle tip bevel that ensures optimum puncturing of the skin. The dot on the protective needle cap is aligned with the needle bevel. This serves as an indication of the bevel position so that the user can adjust the shield to a preferred position without having to remove the protective cap.




Safety Shield
Is pre-assembled directly on the holder and can be positioned in any direction.

VACUETTE® QUICKSHIELD Safety Tube Holder
Item No. 450230

VACUETTE® Multiple Use Drawing Needle
Item No. 450071-77

VACUETTE® VISIO PLUS Needle Transparent View Window
Item No. 450040-43


VISIO PLUS
Transparent view window for optical venipuncture control.

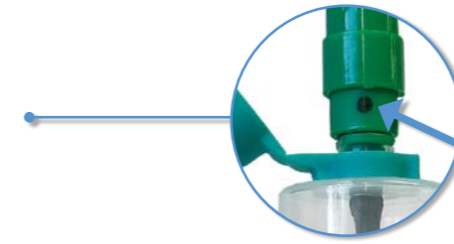

Advantages of the **VACUETTE®** QUICKSHIELD Safety Tube Holder:

- Greater flexibility in regards to user and patient-specific requirements thanks to the rotating safety shield.
- Activated with the aid of a solid surface or thumb.
- Comfort through one-handed activation.
- Safety product suitable for compliance with European Directive 2010/32.
- Dot on the protective needle cap serves as an orientation aid for the needle bevel.
- No changes to collection technique.
- An audible click indicates to the user that the safety shield has been fully activated.
- Can be used with equal ease by both left-handers and right-handers.
- Clear tube holder to differentiate from non-rotatable version.


Instructions for Use *

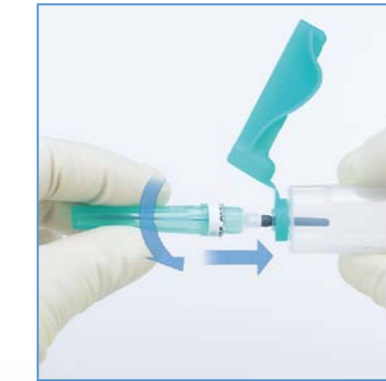

1 Thread the needle



2 If desired, re-position safety shield

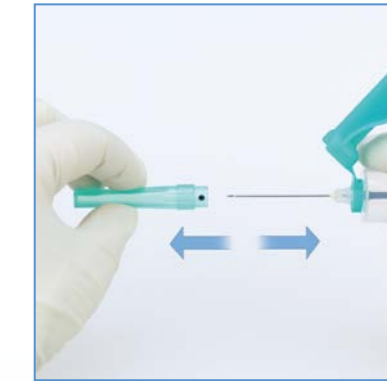

3 Remove protective needle cap



4 Venipuncture



5 Insert the tube



6 Remove the tube



7 Activate the safety mechanism (with the aid of solid surface or thumb)



8 Audible click indicates activation



9 Disposal

The **VACUETTE®** QUICKSHIELD Safety Tube Holder, together with the **VACUETTE®** Blood Collection Tubes and needles, is suitable as a safe single-use system for everyday venous blood collection.

* For the complete handling recommendation please refer to the Instructions for Use for each product.