

# Aesculap Sterile Technology Sterilit®




# Sterilit® our Know-how in instrument care

## Sterilit® – Oil spray

- reduces friction and protects against friction corrosion
- ensures good gliding and lubrication characteristics
- proven physiologically safe in accordance with ISO 10993
- environmentally friendly (CFC-free)
- suitable for steam sterilisation procedures in accordance with ISO 13683 and for hot air sterilisation up to max. 180°C

## Sterilisability

Sterilit®s special composition guarantees penetration of the oil film by the water vapour during the sterilisation process, even in more inaccessible places.

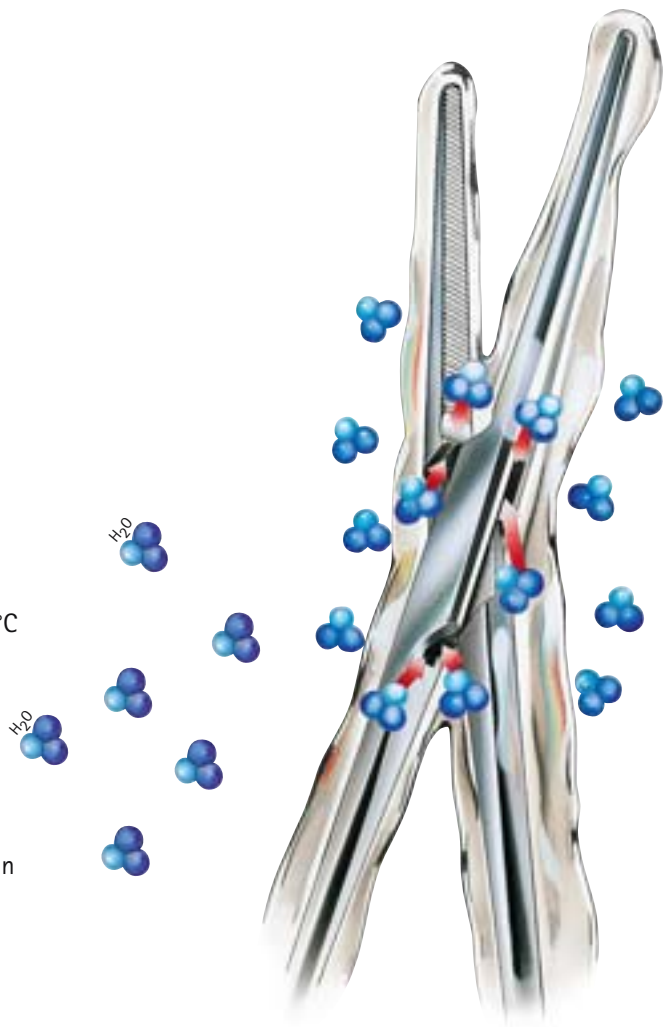

## Aesculap maintenance oils Sterilit® I, Sterilit® Hi, Sterilit® mini, Sterilit® M

### Applications

All Sterilit® maintenance oils are validated for steam sterilization methods according to EN 554/ISO 13683, as well as for dry heat (heated air) sterilization up to 180 °C. They are applied prior to sterilization.

Microbiological efficiency controls for steam sterilization (fractionated vacuum and gravity procedure) have been carried out according to AAMI TIR 12-1994, RKI-Directive, E DIN EN ISO 17664, DIN EN 554 (half cycle method); ANSI/AAMI/ISO 11134, DIN EN 556-1, AAMI/CDV-3 ST 67.

Sterilit®	Art.-No.
Sterilit® I, spray	JG600
Sterilit® I, drip oiler	JG598
Sterilit® Hi, spray	GA536
Sterilit® mini, spray	GB372
Sterilit® M, spray	GB149
Sterilit® M, drip oiler	GA059

### Registration

Sterilit® oils are registered as medical accessory class 1 in conformity with Annex 1 MDD 93/42/EEC CE marking according to directive 93/42/EEC.

### Properties

- Steam-permeable oil film
- No adverse effect on the sterilization performance due to the oil film
- Lubricating effect and corrosion protection
- Silicon-free, no staining and incrustation on the instrument surfaces

### Biological safety – toxicology

There is no evidence that hazards to the patient will arise during surgical application of Sterilit®-treated devices. This is only valid that the Sterilit® oil is properly and carefully used as described in the instructions for use. According to EN ISO 10993-1.

### Safe handling

- Only apply prior to sterilization.
- Store in a cool and dry place protected from light.
- Take note of the use-by date.
- Before using Sterilit, reprocess the object to be lubricated according to the relevant instructions for use.

### Warning

Harmful to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

- Avoid release to the environment.
- Refer to special instructions/safety data sheets.



## What products are available for instrument care?



**STERILIT<sup>®</sup>**

JG600  
including  
2 spray nozzles



Aesculap Oil Spray  
in spray cans (CFC-free),  
for instrument care prior to  
sterilization

**STERILIT<sup>®</sup>**

GB149



Aesculap Oil Spray  
in spray cans (CFC-free),  
especially for the care of  
handpieces with intra-coupling  
prior to sterilization

**STERILIT<sup>®</sup>**

GA536



Aesculap Oil Spray  
in spray cans (CFC-free),  
especially for the care of  
Hilan motors and handpieces  
prior to sterilization

**STERILIT<sup>®</sup>**

*mini*  
GB372



Aesculap Oil Spray  
in spray cans (CFC-free),  
especially for the care of  
miniMotors and handpieces prior  
to sterilization

**STERILIT<sup>®</sup>**

JG598



Aesculap Instrument Oil  
for lubricating joints and locks  
prior to sterilization

**STERILIT<sup>®</sup>**

GA059



Aesculap Motor-Handpiece Oil,  
for lubricating friction  
bearings on motors prior to steri-  
lization



AESCULAP®

**B | BRAUN**  
SHARING EXPERTISE

**Aesculap AG & Co. KG**

Am Aesculap-Platz  
78532 Tuttlingen  
Germany

Phone +49 7461 95-0  
Fax +49 7461 95-2600

[www.aesculap.de](http://www.aesculap.de)

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.