




Medtronic

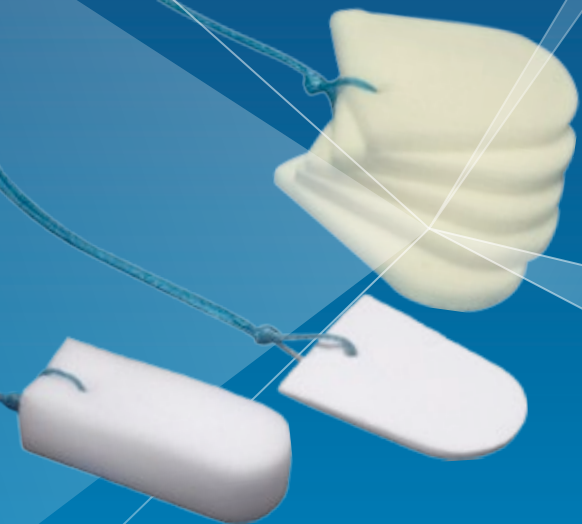

MEROCEL[®]

hemoX[®]

WITH OXIDIZED CELLULOSE, A CLINICALLY
PROVEN HEMOSTATIC AGENT

Innovating for life.

FEEL THE DIFFERENCE WITH MEROCEL[®] *hemoX*[®]

Merocel[®] hemoX[®] with micro-dispersed, oxidized cellulose is an absorbent packing material. When hydrated, it swells to reduce edema and applies gentle pressure to provide hemostatis and absorption of blood and fluids.

The Merocel material is uniformly impregnated with the micro-dispersed, oxidized cellulose. This provides a more lubricious surface that assists in easier removal. Tests have shown increased lubricity during insertion and removal when compared to other Merocel products, as well as reduced post-removal bleeding. The oxidized cellulose also gives an enhanced hemostatis effect. Rehydrated, the sponge can be gently withdrawn using a pair of forceps.

YOUR PATIENTS WILL THANK YOU FOR IT

AVAILABLE IN TWO DENSITIES FOR SITE-SPECIFIC APPLICATIONS

- Open pore cell structure is highly absorbent and fast-wicking, yet strong and durable, soft and comfortable
- Dressings do not shred, tear or leave fiber in the wound



THE MEROCEL VARIETIES

Merocel comes in 3 different types with a complete range of sponges


- hemoX (450-), rapid haemostatis, more lubricant
- Laminated (470-), for easy removal
- Standard Merocel (400-), the original high quality sponge

hemoX

STANDARD POST OP NASAL DRESSINGS

- Ideal for use following septoplasty, rhinoplasty, turbinectomy and other intra-nasal procedures
- Smooth removal
- With or without airway tubes

PRODUCT	DESCRIPTION	LENGTH	THICKNESS	HEIGHT	QTY
450400	hemoX , Standard Nasal Dressing With String	4.5 cm	1.5 cm	2 cm	10
450409	hemoX , Standard Nasal Dressing With String, with Airway	4.5 cm	1.5 cm	2 cm	10
450402	hemoX , Standard Nasal Dressing With String	8 cm	1.5 cm	2 cm	10
450410	hemoX , Standard Nasal Dressing With String, with Airway	8 cm	1.5 cm	2 cm	10

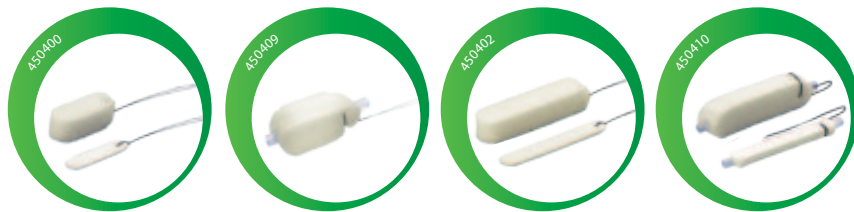


hemoX

ANATOMICAL POST OP NASAL DRESSINGS

- Anatomical nasal dressing ideal for use following intra-nasal procedures
- Expands to fill in the nasal cavity, providing gentle, equal support
- Slim-line 7.7 x 1.0 x 3.3 cm and Standard 7.7 x 1.5 x 3.3 cm

PRODUCT	DESCRIPTION	LENGTH	THICKNESS	HEIGHT	QTY
450413	hemoX, Slimline Doyle Nasal Dressing With String	7.7 cm	1 cm	3.3 cm	10
450424	hemoX, Doyle Nasal Dressing With String	7.7 cm	1.5 cm	3.3 cm	10
450411	hemoX, Doyle Nasal Dressing With String, With AirWay	7.7 cm	1.5 cm	3.3 cm	10

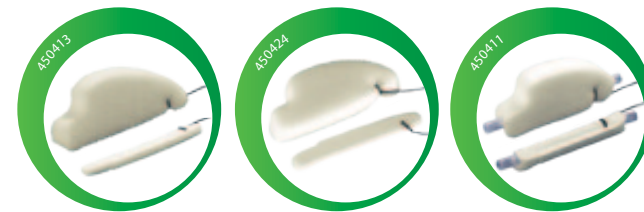


hemoX

EPISTAXIS

MEROCEL HEMOX EPISTAXIS PRODUCTS ARE CONSTRUCTED OF LOW DENSITY, LARGE-PORE MEROCEL SPONGE AND OXIDIZED CELLULOSE. THIS HELPS MAXIMIZE ABSORPTION AND WICKING OF FLUIDS WHILE MINIMIZING PAIN AND REBLEEDING UPON REMOVAL.

Merocel epistaxis products are an ideal use for emergency departments


TIPS FOR THE USE OF EPISTAXIS SPONGES

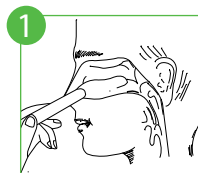

1 Coat With Antibiotic

Remove the Pope Nasal Packaging from its package and coat with watermiscible antibiotic cream. Grasp one end with fingers or forceps.


Drawstring version:
Grasp the strung end.

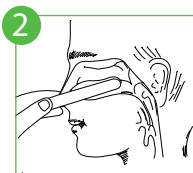

2 Insert Quickly

Quickly advance the packaging along the nasal cavity floor until it reaches the nasopharyngeal wall.


SPEED OF INSERTION IS IMPORTANT.

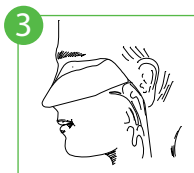

3 Expansion Begins

If the packaging has not fully expanded 30 sec. after placement, it should be irrigated using a syringe with up to 10cc of saline or water.

Drawstring style:
Tape the drawstring over the dorsum of the nose and trim the loose ends.

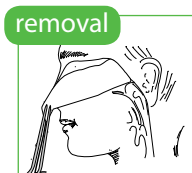

removal




The packaging **MUST** be rehydrated with 10cc of saline or water until saturated. Once rehydrated, grasp the drawstring or the proximal end of the packaging and gently withdraw.

hemoX

EPISTAXIS SPONGES

- Quick & Effective method for treating anterior and posterior epistaxis
- Rapid expansion provides gentle, evenly distributed pressure against the tissues to control bleeding, while platelets aggregate on the surface of the pack to enhance cloth formation
- Anterior bleeding – 5.5cm (Weimert), Posterior bleeding – 10 cm (Pope), Staxi-Stat to fill the contours of the nasal cavity


PRODUCT	DESCRIPTION	LENGTH	THICKNESS	HEIGHT	QTY
450421	hemoX, Weimert Epistaxis Packing, With String	5.5 cm	1.5 cm	2.5 cm	10
450406	hemoX, Pope Epistaxis Packing, With String	10 cm	1.5 cm	2.5 cm	10
450416	hemoX Staxi-Stat Epistaxis Packing, With String	—	—	—	10

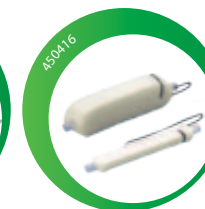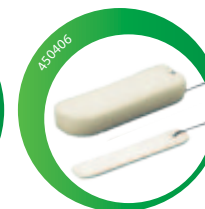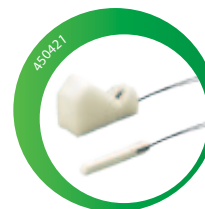

hemoX

SINUS PAKS

THANKS TO OXIDIZED CELLULOSE, THE DRESSING IS DESIGNED TO BE MORE LUBRICIOUS, WHICH MAY PROVIDE A MORE ATRAUMATIC TISSUE REMOVAL AND A REDUCTION IN RE-BLEEDING AS WELL AS AID IN PREVENTION OF ADHESIONS.

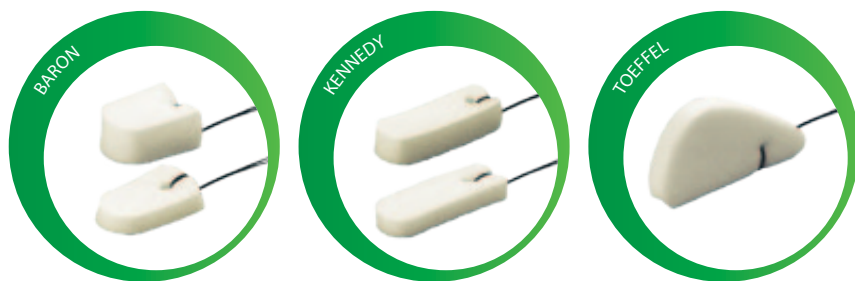

hemoX

SINUS PAKS

- 30% reduction of removal force over other sponges
- Stenting action prevents lateralization of the middle turbinate and aids in prevention of adhesions
- Absorbs up to 20 times its weight in fluids
- Three thicknesses to accommodate various nasal anatomies


PRODUCT	DESCRIPTION	LENGTH	THICKNESS	HEIGHT	QTY
450531	hemoX With String, Baron Ultra-Slim Sinus Pak	2.5 cm	0.6 cm	2 cm	10 pairs
450407	hemoX With String, Baron Regular Sinus Pak	2.5 cm	1.2 cm	2 cm	10 pairs
450530	hemoX With String, Kennedy Ultra-Slim Sinus Pak	3.5 cm	0.6 cm	1.2 cm	10 pairs
450426	hemoX With String, Kennedy Slim Sinus Pak	3.5 cm	0.9 cm	1.2 cm	10 pairs
450422	hemoX With String, Kennedy Regular Sinus Pak	3.5 cm	1.2 cm	1.2 cm	10 pairs
450428	hemoX With String, Toeffel Anatomical Slim Sinus Pak	4.5 cm	0.9 cm	2 cm	10 pairs

hemoX


WICKS AND PACKS

- Wicks are designed for treating otitis externa
- Packs are excellent in absorbing post op fluids
- Soft absorbent sponge to provide comfort and to help keep antibiotics in contact with the ear canal



EAR WICKS AND PACK INSERTION AND REMOVAL

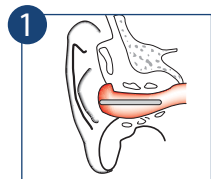

1. Insert into ear canal using ear speculum and forceps.

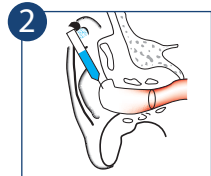

2-3. Instill antibiotic lubricant generously around pack leaving proximal end uncoated for hydration.

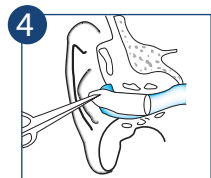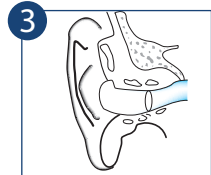

4. Saturate with otic solution for ear wicks and sterile water or saline for ear packs, and wait ten minutes. Grasp with forceps and gently remove. Note: Ear Wick is not recommended for use after post-operative surgery.


For complete instructions for use, please refer to the product packaging

hemoX

POPE EAR WICK

- Ideal to treat otitis externa and for laser-assisted tympanic membrane fenestration procedures
- Expands to fill the ear canal when hydrated with otic solution

PRODUCT	DESCRIPTION	LENGTH	HEIGHT	QTY
450141	hemoX Pope Ear Wick	0.9 cm	1.5 cm	50
450146	hemoX Pope Ear Wick	0.9 cm	2.4 cm	10

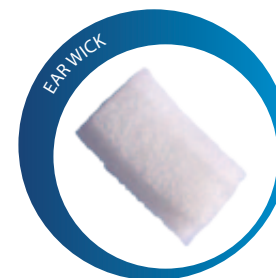

hemoX EAR PACK

- Packs constructed of high density Micropore Meroce sponge with oxidized cellulose. This helps minimize the risk of tissue ingrowth and helps keep the ear canal open while holding the graft in place after middle ear surgery
- Excellent to absorb post op fluids

PRODUCT	DESCRIPTION	DIAMETER	LENGTH	QTY
450356	hemoX Schindler Ear Packing With String	0.9 cm	1.3 cm	10
450355	hemoX Schindler Ear Packing With String	1.2 cm	1.5 cm	10



hemoX SUMMARY TABLE

Want to Switch your standard Meroce sponge to hemoX version?

The Below table will help you to find equivalent hemoX sponge.

MEROCEL hemoX	DESCRIPTION	STANDARD MEROCEL
POST OP		
450400	Nasal Dressing, 4.5 cm	400400
450409	Nasal Pak, 4.5 cm, with Airway	400409
450402	Nasal Dressing, 8 cm	400402
450410	Nasal Dressing with Tube, 8 cm	400410
450413	Doyle Nasal Dressing Slimline	400413
450424	Doyle Nasal Dressing	400424
450411	Doyle Nasal Dressing with Airway	400411
EPISTAXIS		
450421	Weimert Epistaxis Packing	400421
450406	Pope Epistaxis Packing	400406
450416	Staxi-Stat® with Drawstring	400416
SINUS PAKS		
450531	Baron Ultra-Slim Profile	400531
450407	Baron Sinus Pak	400407
450530	Kennedy Ultra Slim Profile	400530
450426	Kennedy Slim Profile	400426
450422	Kennedy Sinus Pak	400422
450428	Toffel Slim Profile	400428
EAR WICKS		
450141	Pope Ear Wick, 15 mm	400141
450146	Pope Ear Wick, 24 mm	400146
EAR PACKS		
450356	Schindler Ear Pack	400356
450355	Schindler Ear Pack	400355

NOTES



A series of horizontal dotted lines providing a template for writing notes. There are 18 dotted lines in total, evenly spaced and extending across the width of the page.



Looking for additional products?

You will find more than 5000 products in our ENT Product Catalog under <http://www.mcatalogs.com/ent/>

www.medtronic.eu

Europe

Medtronic International Trading Sàrl
Route du Molliau 31
Case postale
CH-1131 Tolochenaz
Tel: +41 (0)21 802 70 00
Fax: +41 (0)21 802 79 00

United Kingdom/Ireland

Medtronic Limited
Building 9
Croxley Green Business Park
Hatters Lane
Watford
Herts WD18 8WW
www.medtronic.co.uk
Tel: +44 (0)1923 212213
Fax: +44 (0)1923 241004



Medtronic