



BARRIER[®] Head set

- Flexible draping solution for all head and neck surgery
- Soft and conformable head turban for patient comfort
- Optimal folding for aseptic application, easy for one person to drape

Draping instruction



1. Note the pictogram and keep the drape in the right direction.



2. Unfold until you see three pictograms. One pictogram shows the direction of the drape and the other two show where to put your hands.



3. Put your hands in the corners and unfold the drape.



4. Put the drape under the head and partly under the shoulders of the patient.



5. Take the corner of the drape and unfold over the patient. Pull the drape towards the patient's feet.



6. Do the same on the other side. Cross the drape over the patient.



7. Take one corner of the drape and drape around the part of the face or neck that the procedure involves.



8. Hold the drape tightly to the head and fold over the other side of the drape.



9. Use the adhesive tape to fix the turban in place.



10. Position the adhesive OP sheet centered over the body. Open the sheet towards yourself, then away from you and finally unfold fully towards the feet.



11. Draping is then completed.

Ref. No.	Description	Disp/Trp
696940	Head set 1 mayo stand cover 79x145 cm, 65x85 cm absorbent 1 OP-tape 9x49 cm 1 head drape 150x175 cm and 106x103 cm 1 adhesive OP-sheet 175x175 cm 1 table cover 150x190 cm, 75x190 cm absorbent	6/12
826900	Head drape 150x175 cm, 106x103 cm + 1 OP-tape 9x49 cm	15/30

All specified set contents are also available for inclusion in ProcedurePak® trays.

Ref. No.	ProcedurePak components
988269	Head drape 150x175 cm, 106x103 cm + 1 OP-tape 9x49 cm
969710	Head drape 260x150 cm, split 20x65 cm

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, SE-402 52 Göteborg, Sweden.
T +46 31 722 30 00. F +46 31 722 34 00. www.molnlycke.com

The Mölnlycke Health Care, BARRIER®, and ProcedurePak® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Copyright (2012)