




For light fixation and
comfort after surgery

Suitable and comfortable
during radiotherapy

Celia

COMFORT BRA

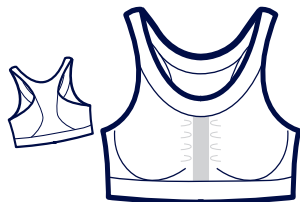




COMPRESSION LEVEL 5 - LIGHT



Breathable



4-way stretch

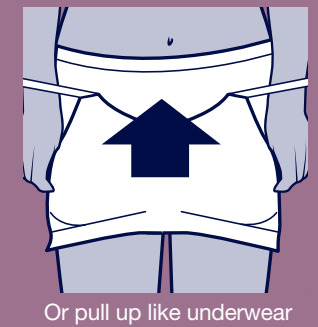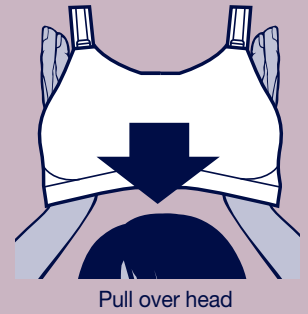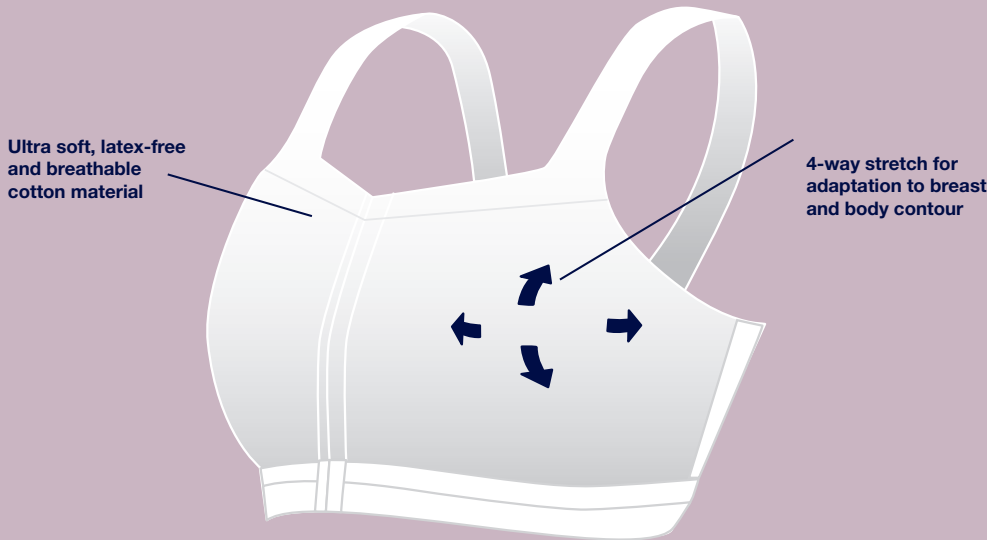

The Celia bra is comfortable to wear when the skin is tender, sensitive and needs protection, e.g. during and after radiotherapy.

It is recommended that the user wears this bra once she has regained full mobility and can lift her arms without any pain or stress on the wound.

It is a comfortable, yet inexpensive choice for light fixation.

Indications:

Medical device for post-surgical use after general surgery, when full mobility is regained. Also suitable for radiotherapy treatment (contains no metal parts).

Precautions:

If you have any questions regarding use and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are placed correctly to avoid pressure points and/or obstructed flow from the drainage tubes. If any unexpected discomfort, wound irritation or similar occurs, please consult your doctor or nurse.

Recommended sizes

Band Size	EU	65 cm	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm	105 cm	110 cm
	US/UK	26 inch	28 inch	30 inch	32 inch	33 inch	35 inch	37 inch	39 inch	41 inch	43 inch
Cup Size	AA										
	A		S								
	B										
	C			M		L		XL			2XL
	D										
	E										

* NOTE: Due to possible post surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size. Important: Always measure yourself to find the right size!

Available colours:



Composition:

60 % Cotton
25 % Polyamide
15 % Elastane

Care Handling:

Coloured:



104°F

White:



140°F

Wash with similar colours in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Dealer: