


Product Specifications

Brand: Curion® CuriSheath

Product: Silicone External Catheter

Product description

The Curion® CuriSheath is a non-sterile external catheter designed for non-invasive management of penile urinary incontinence. It comes in three variants: standard, active and pop-on, with five diameter sizes for each type. The standard CuriSheath has an unrolled sheath (C) length of 80mm with an adhesive region of 40mm. The active CuriSheath is equivalent to the standard CuriSheath, but with a 75% greater adhesive region, designed to provide added security for patients. The pop-on CuriSheath has a shorter length sheath, designed for a retracted penis.

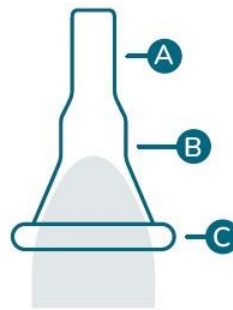

The Curion® CuriSheath is made of soft, transparent silicone and contains an integral medical grade adhesive within the proximal unrolled sheath (C) for self-adhesion to the penis. The distal end of the catheter consists of a spout (A) which can be connected to a Curion® CuriBag® for urine drainage and collection. The Curion® CuriSheath also features a reservoir (B) which maintains a steady flow output and prevents urine inflowing back from the tubing to the penis. Once affixed, the Curion® CuriSheath can remain in place for up to 24 hours.

Intended use

The Curion® CuriSheath is intended to collect and channel urine from the penis to a urine collection bag.

Classification

Class I under the Medical Devices Regulations (SI 2002 No. 618, as amended) and
Class I under Regulation (EU) 2017/745 Medical Device Regulation (MDR).

Product Specifications

Precautions

The Curion® CuriSheath must not be used if an allergic reaction occurs. It is for single-use only and reuse may create risk of leakage due to poor adhesion. Always check if skin is irritated between applications and do not use if skin is irritated. Ensure the correct size is used to prevent restriction of blood flow (too tight) and leakage (too loose).

Product Reference & Description

Product Reference	Product Description	Type (Colour Code)	Sheath Diameter* (mm)
CSTD25	Curion CuriSheath self-adhesive silicone external catheter standard version 25mm	Standard (Green)	25
CSTD29	Curion CuriSheath self-adhesive silicone external catheter standard version 29mm	Standard (Purple)	29
CSTD32	Curion CuriSheath self-adhesive silicone external catheter standard version 32mm	Standard (Yellow)	32
CSTD36	Curion CuriSheath self-adhesive silicone external catheter standard version 36mm	Standard (Blue)	36
CSTD41	Curion CuriSheath self-adhesive silicone external catheter standard version 41mm	Standard (Red)	41
CACT25	Curion CuriSheath self-adhesive silicone external catheter active version 25mm	Active (Green)	25
CACT29	Curion CuriSheath self-adhesive silicone external catheter active version 29mm	Active (Purple)	29
CACT32	Curion CuriSheath self-adhesive silicone external catheter active version 32mm	Active (Yellow)	32
CACT36	Curion CuriSheath self-adhesive silicone external catheter active version 36mm	Active (Blue)	36
CACT41	Curion CuriSheath self-adhesive silicone external catheter active version 41mm	Active (Red)	41
CPOP25	Curion CuriSheath self-adhesive silicone external catheter pop-on version 25mm	Pop-on (Green)	25
CPOP29	Curion CuriSheath self-adhesive silicone external catheter pop-on version 29mm	Pop-on (Purple)	29
CPOP32	Curion CuriSheath self-adhesive silicone external catheter pop-on version 32mm	Pop-on (Yellow)	32
CPOP36	Curion CuriSheath self-adhesive silicone external catheter pop-on version 36mm	Pop-on (Blue)	36
CPOP41	Curion CuriSheath self-adhesive silicone external catheter pop-on version 41mm	Pop-on (Red)	41

*Sizes are approximate

Packaging Information

Product Reference	Inner Carton Qty	Outer Carton Qty	EAN Eaches	EAN Inner Carton	EAN Outer Carton
CSTD25	30	180	8715343742235	8715343742730	8715343742723
CSTD29	30	180	8715343742228	8715343742716	8715343742709

Product Specifications

CSTD32	30	180	8715343742211	8715343742693	8715343742686
CSTD36	30	180	8715343742204	8715343742679	8715343742662
CSTD41	30	180	8715343742198	8715343742655	8715343742648
CACT25	30	180	8715343742181	8715343742631	8715343742624
CACT29	30	180	8715343742174	8715343742617	8715343742600
CACT32	30	180	8715343742167	8715343742594	8715343742587
CACT36	30	180	8715343742150	8715343742570	8715343742563
CACT41	30	180	8715343742143	8715343742556	8715343742549
CPOP25	30	180	8715343742136	8715343742532	8715343742525
CPOP29	30	180	8715343742129	8715343742518	8715343742501
CPOP32	30	180	8715343742112	8715343742495	8715343742488
CPOP36	30	180	8715343742105	8715343742471	8715343742464
CPOP41	30	180	8715343742099	8715343742457	8715343742440



Specifications

Product Reference	Material Composition	Sheath Diameter (mm)	Sheath Length (mm)	Adhesive Length (mm)	Spout Length (mm)
CSTD25	Liquid silicone rubber (medical grade), solvent based acrylic emulsion adhesive layer (medical grade), transparent	25	80	40	30
CSTD29		29			
CSTD32		32			
CSTD36		36			
CSTD41		41			
CACT25		25	80	70	
CACT29		29			


Product Specifications

CACT32		32	45	40	
CACT36		36			
CACT41		41			
CPOP25		25			
CPOP29		29			
CPOP32		32			
CPOP36		36			
CPOP41		41			
Sombrero holder (Active & Pop-on SKUs only)	High Impact Polystyrene (HIPS), white	-	-	-	-


*measurements may vary by ± 5%

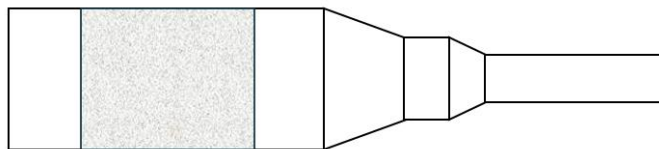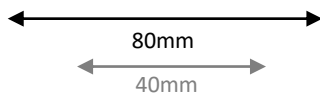

Standard



Adhesive Region 

Images are for illustrative purposes only

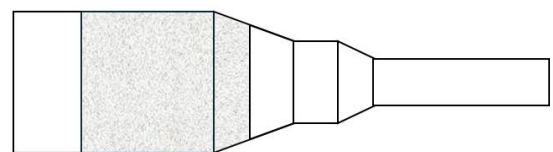

Pop-On

30mm

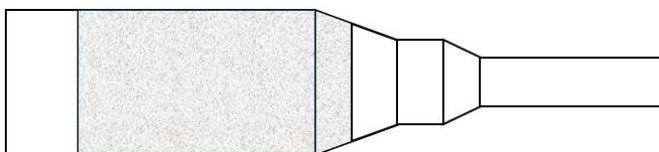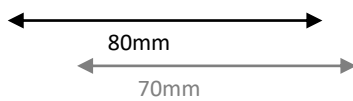

Active

Product Specifications

Absence of specific components

Component	Present
Latex	No
Phthalates	No
Substances of animal origin	No
Substances of human origin	No
Blood or blood derivatives	No
Medicinal substances	No
Heavy metals	No

Sterilization

The Curion® CuriSheath is non-sterile.

Storage conditions

The Curion® CuriSheath should be stored in a cool well-ventilated and dry place away from direct sunlight.

Shelf life

The Curion® CuriSheath has a shelf life of five (5) years from the production date, when stored under specified conditions.

Product Specifications

Pictures



CuriSheath Standard



CuriSheath Active / Pop-on



(From L-R): Standard, Active, Pop-On

Approval	Name	Date	Signature
Product Manager	Jeston D'Costa	02/03/2026	
RA Specialist	Federica Russo	02/03/2026	