



REUSABLE
DESIGN



Single patient-use ENFit™ syringes

Cenaman® ENFit syringe – single patient use are syringes used for delivering liquid medication and enteral nutrition orally or via a feeding tube into the gastro-intestinal tract and hydration.

These syringes can be reused in the same patient up to 14 days or 84 usages (whichever comes first).

By allowing multiple uses on the same patient, these syringes reduce medical waste and contribute to a more sustainable healthcare practice.



Classification

The Cenaman ENFit™ syringes reduce the risk of an unintended misconnection. All syringes are equipped with the safe enteral connector ENFit™, according to international safety standard ISO 80369-3, thus offering the highest safety in the clinical practice.

Class Is under the Medical Devices Regulations (SI 2002 No. 618, as amended) and Class Is under Regulation (EU) 2017/745 Medical Device Regulation (MDR).

Packaging and product reference

REF code	Size	description	pcs per inner box	pcs per outer box
C063001	1 ml	Cenaman Multi-use EnFit syringe 1 ml	10 (10 x 1)	1200 (120x10)
C063002	2,5 ml	Cenaman Multi-use EnFit syringe 2,5 ml	10 (10 x 1)	1200 (120x10)
C063005	5 ml	Cenaman Multi-use EnFit syringe 5 ml	10 (10 x 1)	1200 (120x10)
C063010	10 ml	Cenaman Multi-use EnFit syringe 10 ml	10 (10 x 1)	800 (80x10)
C063020	20 ml	Cenaman Multi-use EnFit syringe 20 ml	10 (10 x 1)	360 (36x10)
C063020-30	20 ml	Cenaman Multi-use EnFit syringe 20 ml x30	30 (30 x 1)	480 (16x30)
C063061	60 ml	Cenaman Multi-use EnFit syringe 60 ml eccentric	10 (10 x 1)	240 (24x10)

REF code	Size	description	EAN inner box	EAN outer box
C063001	1 ml	Cenaman Multi-use EnFit syringe 1 ml	16971564466483	26971564466480
C063002	2,5 ml	Cenaman Multi-use EnFit syringe 2,5 ml	16971564466490	26971564466497
C063005	5 ml	Cenaman Multi-use EnFit syringe 5 ml	16971564466506	26971564466503
C063010	10 ml	Cenaman Multi-use EnFit syringe 10 ml	16971564466513	26971564466510
C063020	20 ml	Cenaman Multi-use EnFit syringe 20 ml	16971564466520	26971564466527
C063020-30	20 ml	Cenaman Multi-use EnFit syringe 20 ml x30	56971564461721	66971564461728
C063061	60 ml	Cenaman Multi-use EnFit syringe 60 ml eccentric	16971564466537	26971564466534

Features

- EnFit connector (ISO 80369-3)
- Clear number markings
- High visibility through barrel
- Purple colored plunger
- Low dose tip in 1ml, 2.5ml and 5ml
- Smooth plunger movement
- Sterile (EO sterilized)



Instructions for use and reuse

Do not autoclave or sterilize syringe as this may cause syringe to malfunction. Do not place syringe in dishwasher as this may cause syringe to malfunction.



1. Remove plunger from syringe barrel.
2. Wash the barrel and plunger separately in warm soapy water until visible dirt is removed. Follow local guidelines on the appropriate soap.
3. If visible soil remains, repeat cleaning steps using a soft-bristled brush to aid in washing the plunger and the INSIDE of the syringe barrel. Do not use abrasive products on the OUTSIDE of the barrel as this may lead to premature removal of printed markings on the syringe.
4. Rinse barrel and plunger with cold running tap water until no traces of soap remains.
5. Shake or/and absorb with gauze any residual water of the syringe components and then allow both parts to air dry in a clean area before putting plunger back in barrel. Repeat cleaning steps if visible soil remains.
6. Store in a clean dry container with lid.

Absence of specific components

Free of latex, phthalates, substances of animal and/or human origin, blood or blood derivatives, medicinal substances, heavy metals and PVC.

Shelf life

Cenaman® ENFit syringe – single patient use have a shelf-life of 5 years from production date, when stored under specified conditions.