

Soluscope™ PAA



Disinfectant for automated endoscope reprocessing

Disinfectant for flexible endoscopes to be used exclusively with Soluscope Serie 4 PA, Serie XS and Serie 5

- Peracetic acid-based disinfectant
- Broad spectrum antimicrobial activity
- Compliant with EN ISO 15883-4
- Compatible with a large range of flexible endoscopes
- Lock key system and cannula integrated to the container



Soluscope™ PAA

Disinfectant for flexible endoscopes to be used exclusively with Soluscope Serie 4 PA, Serie XS and Serie 5



Soluscope Serie 4 PA



Soluscope Serie XS



Soluscope Serie 5

Product composition

- 5% (w/w) Peracetic Acid
- Stabilising agents

Product characteristics

- Shelf life: 24 months closed
- Aspect: clear and colourless
- Odour: characteristic
- pH pure product: 1 - 4.2*
- Specific gravity at 20°C: 1.10 - 1.13
- Refractive index: 1.35 - 1.37
- Viscosity: < 10 mPa/s

*pH value measurements under operating conditions by means of a pH meter with temperature compensation.

Microbiological properties

| Claim/Conditions | Concentration | Contact time | Temperature |
|---|---------------|--------------|-------------|
| Soluscope Serie 4 PA (in WD) and Soluscope Serie 5 (in WD) | | | |
| Bactericidal* (EN 13727, EN 14561), Mycobactericidal* (EN 14348, EN 14563), Fungicidal* (EN 13624, EN 14562) & Virucidal* (EN 14476, EN 17111) | 1.8% | 3 mins | 40°C |
| Soluscope Serie XS (in WD) | | | |
| Bactericidal* (EN 13727, EN 14561), Mycobactericidal* (EN 14348, EN 14563), Fungicidal* (EN 13624, EN 14562) & Virucidal* (EN 14476, EN 17111) | 1.0% | 6 mins | 40°C |

Refer to the washer disinfectant's manual for conditions for use

*=clean conditions

Packaging and ordering information

| Product | Packaging unit | Order code |
|---------------|----------------|------------|
| Soluscope PAA | 2 x 5 L | xxxxx |

Instructions for use

Concentrated solution

- Screw the red-capped plunger onto the container. Make sure that the vent hole is not blocked
- Put the container in place and connect it to the WD (refer to WD user manual)
- Automated dilution for WD

Shelf life after opening: 2 months while connected to the WD (write the opening date on the label in the location provided for this purpose).


Storage

Warning – respect the precautions for use (drawn up according to the European rules in force regarding the classification and labelling of chemical products). Store between +5°C and +25°C.

Medical device use safely

Class IIb

Please read label and product information.

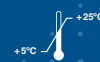

FOR PROFESSIONAL USE ONLY.