

Product Specifications

Brand : Klinion®
Product : Blood Collection Safety Needles (with & without holder)

Product description

Klinion® Blood Collection Safety Needles are disposable needles for the collection of blood, with safety shield activation incorporated. After the withdrawal of the needle from the body, the attached needle safety shield can be manually activated to cover the needle immediately after use to minimize risk of accidental needle stick, protecting both healthcare professionals and patients. Different needle sizes are available, with or without holder. Compatible with standard holder.

Intended use

Klinion® Blood Collection Safety Needles are intended to be used with vacuum blood collection tube for the collection of venous blood, by healthcare professionals.

Classification

Class IIa under the Medical Devices Regulations (SI 2002 No. 618, as amended) and
Class IIa under Regulation (EU) 2017/745 Medical Device Regulation (MDR).

Packaging & Product Reference

Product Reference	Description	Colour Code	Needle Gauge	Needle Length	Piece (s) per Peel Pouch	Pieces per Inner Carton	Pieces per Outer Carton
C0880400	KLINION SAFETY BL. COL. NEEDLE 21G 32 MM	Green	21G	32MM	N/A	50	500 (10 x 50)
C0880401	KLINION SAFETY BL. COL. NEEDLE 22G 32 MM	Black	22G	32MM	N/A	50	500 (10 x 50)
C0880410	KLINION BL. COL. NEEDLE HOLDER 21 G	Green	21G	32MM	1	50	500 (10 x 50 x 1)
C0880411	KLINION BL. COL. NEEDLE HOLDER 22 G	Black	22G	32MM	1	50	500 (10 x 50 x 1)

Product Specifications

Product Reference	Description	Colour Code	Needle Gauge	Needle length	EAN Peel Pouch	EAN Inner Carton	EAN Outer Carton
C0880400	KLINION SAFETY BL. COL. NEEDLE 21G 32 MM	Green	21G	32MM	N/A	16971564462485	26971564462482
C0880401	KLINION SAFETY BL. COL. NEEDLE 22G 32 MM	Black	22G	32MM	N/A	16971564462492	26971564462499
C0880410	KLINION BL. COL. NEEDLE HOLDER 21 G	Green	21G	32MM	06971564462549	16971564462546	26971564462543
C0880411	KLINION BL. COL. NEEDLE HOLDER 22 G	Black	22G	32MM	06971564462556	16971564462553	246971564462550

Features

- Color coded for easy identification of needle sizes
- With safety shield activation incorporated, for needle stick prevention.
- Knock or thump push to activate safety mechanism
- Safety cover integrated with the cap.
- Compatible with standard holder.

Contra-Indications

Do not use Klinion® Blood Collection Safety Needles for other than blood collection.

Precautions

- Venipuncture should be performed wearing gloves.
- Do not use if packaging is damaged or not intact.
- Do not use after expire date.
- Do not re-sterilize.
- If the needle is blocked or damaged, please do not use it.
- Rough handling of the device may cause damage to the device.
- Do not forcefully release or destroy the accidental needlestick prevention mechanism after it has been activated.
- While the safely mechanism is being activated, keep hands away from the needle tip at all times.
- There is a possibility of blood or other fluids leaking out of the product. when removing the luer connector. Handle with precaution.
- Due to the risk of blood dripping from the exposed tip, promptly dispose of the product pointing the needle lip upward in an approved disposal container after activating the safety mechanism.

Product Specifications

- While using and during post-use disposal, please pay attention to prevent accidental needlestick.
- If you puncture the tube with the rubbered needle more than 5 times, there is risk of leakages. It is therefore recommended that the device be used in a maximum of 5 sampling in order to achieve the expected performance.
- Do not re-use, for single use-only. Re-use may lead to infection or other illnesses.
- Handle all biologic samples and blood collection "sharps" (lancets, needles, luer adapters, and blood collection sets) in accordance with the policies and procedures of your facility.
- Obtain appropriate medical attention in the event of any exposure, as body fluids may transmit viral hepatitis, HIV(AIDS) or other blood borne pathogens.
- For use under the guidance of a healthcare professional.

Presence of specific components

Component	Present
Latex	No
Phthalates	No
Substances of animal origin	No
Substances of human origin	No
Blood or blood derivates	No
BPA	No
Heavy metals	No

Sterilization

EO Sterilized.

Storage conditions

Klinion® Blood Collection Safety Needles should be stored in a cool, dry environment away from direct sunlight, high heat, and high humidity.

Shelf life

Klinion® Blood Collection Safety Needles have a shelf-life of 5 years from production date, when stored under specified conditions.

Product Specifications

Picture



Product ref.: C0880400



Product ref.: C0880410

Approval	Name	Date	Signature
Associate Category Management Director	Susanne van den Berg		
RA Specialist	Alexandra Maria da Silva Duarte		